
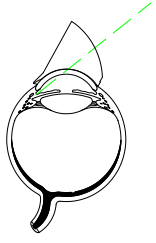


## Ocular Secureflex HF Surgical Gonio Lens

 UK CA CE 1639	Product Code	Gonio Mag	Static Field of View	Contact OD	Lens Height	
	OSIG	1.20x	90°	15.5mm	14.3mm	

*Caution: Federal law restricts this device to sale by or on the order of a physician.*

### Intended Use

The SecureFlex HF Gonio lens is intended to be placed on the eye of a patient to allow a physician to view the anterior structures of the eye during examination and therapy of intraocular abnormalities.

### Indication for Use

The SecureFlex HF Gonio lens is indicated for use during goniotomy and other anterior segment surgeries. The SecureFlex provides a clear view of the anterior chamber angle while viewed through a surgical microscope.

### Design Features

- Ideal for anterior chamber angle surgeries
- Hands Free self retaining design
- Resists “decoupling” of optical interface during surgery
- Ample access to clear cornea
- Allows for Bimanual surgery
- Flexible lens flange for stability
- Designed to be used once, then discarded, eliminating the risk of cross patient contamination.
- Packaged individually in sterile peel pack.

### Method of Use

Rinse the lens with sterile water before use. Place a few drops of an appropriate viscoelastic coupling fluid on the contact surface of the lens prior to placing the lens on the patients eye. A combination of eye and microscope rotation is required for proper viewing of the anterior chamber angle. The resultant angle is approximately 30 degrees from the axis of the patients eye. Lenses are to be used once and discarded.

### Warnings

Do not use if there is damage to lens or sterile package.

### Contraindications

The Ocular SecureFlex HF Gonio lenses are is intended for transient use only.

